



Communication Solutions

User Manual

www.artonecs.com

Artone MS Bluetooth Multipoint Stereo Loopset

User Manual

- 1 – Overview
- 2 - Getting started
- 3 - Pairing
- 4- Using Artone MS

Product Information

Features

Artone MS is a portable wireless listening assistive device with built-in latest Bluetooth wireless technology.

It offers handsfree functionality for mobile phones, in addition it allows stereo music streaming from your Bluetooth enabled devices.

Compatibility

Artone MS is compatible with all hearing aids equipped with t-coil function as well with cellular phones or other Bluetooth enabled devices that support Bluetooth Headset, Hands free, A2DP and AVRCP profiles.

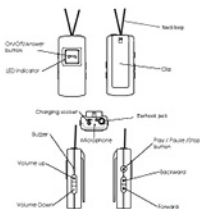
Working Range

Artone MS conforms to Bluetooth class 2 for a range of 30 feet (10M) in an open space.

Working Time

Music playback time: 10 hours Talk time: 12 hours.

1. Artone MS Overview



Artone. All Rights Reserved.

2. Getting Started

Charging Your Artone MS

Artone MS is equipped with a lightweight and durable Lithium Polymer rechargeable battery and must be charged prior to use.

- Please plug the supplied charger into the charging socket of the unit.
- The Red LED indicator will turn on while charging. average charging time 3 hours.

LED Indicator

- Blue light flashes: Power on / Standby / Working mode
- Blue/Red light flash alternately: Pairing mode
- Red light flashes continuously: Low battery mode

Turning Your device On/Off

- Turn on: Long press the button for 4 seconds and the blue LED indicator flashes.
- Turn off: Long press the button for 4 seconds and the red LED flashes once and go off.

Turning the Built-In Buzzer On/Off

Artone MS has built-in buzzer and it's set to "ON" when shipped. When you turn the buzzer on, you can hear "Beep!" from the device when there's an incoming call.

Otherwise, you can hear the ring tone from the hearing aid only (while on t-mode).



Turn on the buzzer:

- When the buzzer is OFF. Under standby mode, press and hold the both volume buttons at the same time until you hear the “Do Re Mi” tone from the hearing aid (while on t-mode).

Turn off the buzzer:

- When the buzzer is ON. Under standby mode, press and hold the both volume buttons at the same time until you hear the “Mi Re Do” tone from the hearing aid (while on t-mode).

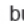
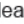
Controlling Your device Volume

- Increase Volume: Short press the button 
- Reduce Volume: Short press the button 

To ensure you have optimal volume please adjust the volume of your phone to maximum.

3. Pairing

Pairing Artone MS Loopset with your phone or other Bluetooth® devices.

- Please turn the Artone MS off. (from OFF state)
- Press and hold the  button until both Blue and Red light indicators flash alternately, then release the  button.
- Perform a Bluetooth® device discovery from your phone or other Bluetooth devices and you will see the device ID “Artone MS” displayed on your device’s screen. Select “Artone MS” and enter the default password “1234” to start pairing. After the pairing is successful, the red light indicator will go off and blue light indicator will remain flashing.

3.2 Multipoint function

(Can connect with 2 Bluetooth® devices at the same time)

Artone MS supports Multipoint technology, which permits simultaneous connections with two different Bluetooth® devices. To pair the 2nd Bluetooth® device, please repeat the pairing step 3. When pairing with TVB please perform search on the TVB and repeat step 3 from this manual.


- Working with TV - pair the TVB with Artone MS, plug the TVB in your TV headphone connection (or other connection supplied), then press multifunctional button on TVB to start streaming the audio signal.

* In order to receive calls while connected to TVB you must pair your Artone MS to Bluetooth® mobile phone.

- **To answer call when you are watching TV call**

If you choose to answer call, press the  button to pick up the call.

- **To reject call when you are listening to TV**

If you would not answer the call; please long press the  button to reject it.

- **You can press the Forward button  to switch between calls.**

3.3 Using Artone MS with PCs

If your PC or laptop is equipped with Bluetooth® capability, then you can use Artone MS to listen to music or make internet call on Skype™/MSN®/Yahoo® messenger. Please activate your Bluetooth® software first and refer to pairing step 3 to pair Artone MS Loopset with your Bluetooth® devices. For the details of operation, please refer to the user guide of the Bluetooth® software installed on your PC or laptop.

4. Using Your Artone MS

4.1 Hearing aid –


Switch your hearing aid to T-mode/ or MT-mode. (telecoil)

4.2 Using your Artone MS device with phones

- **Making a call**

Dial the phone number from your phone.


- **Receive a call**

When there's an incoming call, you will hear a "Beep!" sound from the Artone MS buzzer or a ring tone from the hearing aid. Just short press the  button to pick up the call.


- **End a call**

Short press the  button to end a call.

- **Reject a call**


When your phone rings or hear a "Beep!" sound from the Loopset buzzer or a ring tone from the hearing aid (while on t-mode), just press the  button for 3 seconds and release it when you hear a "Beep!" sound to reject a call.

Make a speed dial for a pre-stored phone number


On standby mode, press and hold the  button for about 2 seconds and release it until you hear a “Beep!” sound from the hearing aid (while on t-mode). Artone MS will auto dial the pre-stored phone number.

- **Do not hold the button more than 4 seconds to avoid turning off the device.**

Redial the last number



On standby mode, short press the backward button  and release it until you hear a “Beep!” sound from the hearing aid (while on t-mode). Artone MS will auto redial the last number.

Make a voice dialing

On standby mode, short press the forward button . When you hear a “Beep!” sound from the hearing aid (while in t-mode), just speak the voice tag to the MIC of the device. Then your phone will auto dial out the number.

- **Please ensure to save the voice tag into your phone first before using the voice dialing function.**

Retrieve calls in phone

When the Artone MS is on a phone call, short press the backward button  to transfer the call to your phone. If you would change the call from your phone to the Artone MS during conversation, just short press the backward  button again.

Listen to music from your phone

You can use Artone MS Loopset to listen to stereo music streamed from your phone and remote control music if your phone supports Advanced Audio Distribution Profile (A2DP) and AVRCP Profile.

Remote control function (AVRCP)

- **Play music**

Press play button to start play (*Some phones may require you to start a media player first)

- **Pause music**

Press to pause music.

- **Stop music**

Long press button to stop music.



- **Backward / Forward songs**

Backward, jump to the previous song: Press the button ⏮

Forward, jump to the next song: Press the button ⏭

- **The operation of A2DP/AVRCP is different from diverse phones; please refer to your phone's user guide for detail operation.**

Answering a call when listening music

Press the  button once to answer the call and press it again to end the  call and resume the music. If the other party you are speaking to end the call first, the music will automatically resume without having to press the button. If you can't resume the music automatically, please short press the play button to resume it.

- **Some phones don't support Reject Calls, Last Number Dialing, Voice Dialing, Retrieve calls in phones and A2DP/AVRCP. For details, please refer to your phone's user guide.**

Warning:

Do not use near machinery as the loop may be caught up in the moving parts.

Please express extra caution while using the Loopset not to get caught on anything as it may cause injuries.

DO not use the Loopset if you use a pace maker or any other devise that may be sensitive to magnetic/inductive field.

* parts of the Artone Bluetooth Loopset may be magnetic, please do not place near the BT Loopset near credit card or magnetic storage media as the information may be erased.

* express extra caution while driving, and obey your state/county laws regarding usage of mobile phone while driving.

Keep away from children.

Warranty

Artone, as defined below, warrants this Artone-branded product against defects in materials and workmanship under normal use for a period one year from the date of retail purchase by the original end-user purchaser ("Warranty Period"). If a product defect arises and a valid claim is received within the Warranty Period, at its consideration, Artone will either exchange the product with a new one or if possible repair the product.

The warranty does not cover abuse, deliberate damage, negligent use, and any damage caused exceeding the lines of "normal" product use.

All claims must be presented with the invoice of purchase and this document with a stamp of the manufacturer.

Warranty Limitations

Artone limited warranty provides that, subject to the following limitations, each Product will be free from defects in material and workmanship and will conform to Artone specification for the particular Product.

LIMITATION OF REMEDIES

YOUR EXCLUSIVE REMEDY FOR ANY DEFECTIVE PRODUCT IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT.

Artone may elect which remedy or combination of remedies to provide in its sole discretion. Artone shall have a reasonable time after determining that a defective Product exists to repair or replace a defective Product. Artone's replacement Product under its limited warranty will be manufactured from new and/or serviceable used parts. Artone's warranty applies to repaired or replaced Products for the balance of the applicable period of the original warranty.

LIMITATION OF DAMAGES

ARTONE'S ENTIRE LIABILITY FOR ANY DEFECTIVE PRODUCT SHALL IN NO EVENT EXCEED THE PURCHASE PRICE FOR THE DEFECTIVE PRODUCT. THIS LIMITATION APPLIES EVEN IF ARTONE CANNOT OR DOES NOT REPAIR OR REPLACE ANY DEFECTIVE PRODUCT AND YOUR EXCLUSIVE REMEDY FAILS OF ITS ESSENTIAL PURPOSE. NO CONSEQUENTIAL OR OTHER DAMAGES NOTWITHSTANDING ANYTHING ELSE IN THIS POLICY OR OTHERWISE, WILL NOT BE LIABLE WITH RESPECT TO THE PRODUCTS UNDER ANY CONTRACT, NEGLIGENCE, STRICT LIABILITY OR OTHER LEGAL OR EQUITABLE THEORY (I) FOR ANY AMOUNT IN EXCESS OF THE PURCHASE PRICE FOR THE DEFECTIVE PRODUCT OR (II) FOR ANY GENERAL, CONSEQUENTIAL, PUNITIVE, INCIDENTAL OR SPECIAL DAMAGES. THESE INCLUDE LOSS OF LOST PROFITS, THE INSTALLATION OF REPLACEMENT PRODUCTS, AND ANY INSPECTION, TESTING, ANY DEFECT OR BY THE REPAIR OR REPLACEMENT OF PRODUCTS ARISING FROM A DEFECT IN ANY PRODUCT. THERE ARE NO WARRANTIES WHICH EXTEND BEYOND THE FACE OF THE ARTONE LIMITED WARRANTY. ARTONE DISCLAIMS ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, REGARDING THE PRODUCTS, INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR NONINFRINGEMENT.